



A NEW ERA FOR DRUG TESTING

SOTOXA™ ORAL FLUID MOBILE TEST SYSTEM



THC
25 NG/ML



COC
30 NG/ML



Rapid drug screening
is in your hands

KEY FEATURES



PORTABLE

- Handheld, lightweight and compact
- Carrying case stores analyser, printer and all components
- Optional car charging kit is available



QUALITY CONTROL

- When the analyser is turned on, start-up checks are automatically activated
- Positive and negative quality control cartridges (PQC and NQC) check that the analyser is interpreting test lines correctly
- QC testing can be enforced prior to each test, every day, week or two weeks
- On-board sensors monitor test progress
- On-board heater ensures tests run at the optimal temperature, reducing variability



EASY TO USE

- 3-step testing process
- Each test cartridge has a Quick Response (QR) code to reduce user error
- The simplicity of the device minimises user training requirements, allowing fast implementation
- Analyser interprets and determine the test outcome, eliminating the subjectivity of visually interpreted results



DATA MANAGEMENT

- Internal storage capacity of over 10,000 tests
- Printer is provided



FAST RESULTS

- Collection device designed for active swabbing which speeds up collection
- Sample collection in 60 seconds or less¹
- Volume adequacy indicator turns blue when enough sample has been collected. No timed collection
- Results in 5 minutes

TEST OPTIONS

DRUG CLASS	ABBREVIATION	TARGET DRUG	CUTOFF (NG/ML)
Amphetamine	AMP	(S) Amphetamine	50
Benzodiazepines	BZO	Temazepam	20
Cannabis (THC)	THC	Δ^9 THC	25
Cocaine metabolite	COC	Benzoyllecgonine	30
Methamphetamine	MAMP	(S) Methamphetamine	50
Opiates	OPI	Morphine	40

SOTOXA™ ORAL FLUID MOBILE TEST SYSTEM

TESTING WHERE IT IS NEEDED

Substance abuse can result in low productivity, accidents, injuries and poor health. An effective, simple and fast drug screening solution is key for both deterrence and identifying if drugs are being used.

INCREASE SAFETY

Keeping our communities safe is a key priority. The SoToxa handheld technology with immediate actionable results can improve safety and help individuals live their best lives.

PORTABLE, HANDHELD AND CONVENIENT

The handheld SoToxa analyser provides the optimal combination of speed, ease of use, reliability and accuracy, empowering the users to take action with immediate test results.





TESTING WHERE YOU NEED IT

The SoToxa Oral Fluid Mobile Test System is the ideal instrument for drug testing in those often difficult environments. Samples can be collected anywhere without the need for gender-specific facilities. It can detect up to 6 drug classes within 5 minutes from a single oral fluid collection sample. Weighing in at under 700 g and fitting comfortably in your hand, it is a truly portable analyser. The test system is designed to operate in a wide range of environments. The backlit, 3.2 inch full colour screen provides excellent visibility in daylight and at night.

HOW IT WORKS

1

**INSERT THE TEST CARTRIDGE
INTO THE ANALYSER**

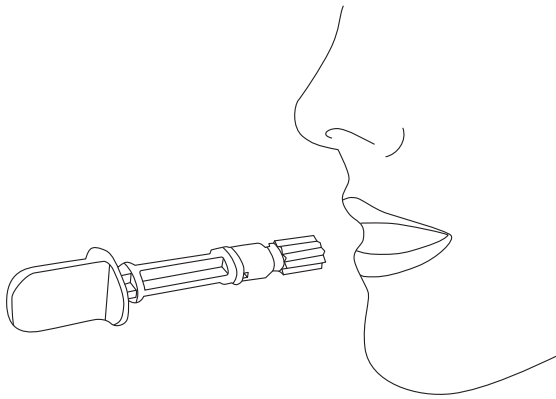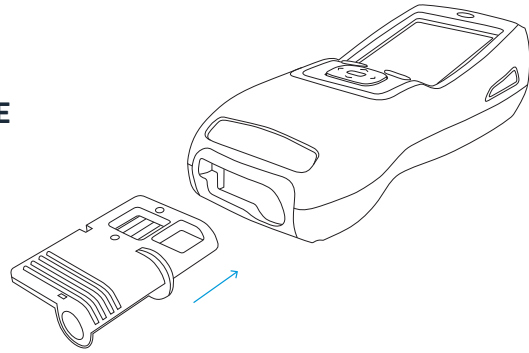


2

**COLLECT THE
ORAL FLUID SAMPLE**

3

**INSERT THE COLLECTION DEVICE
INTO THE TEST CARTRIDGE**

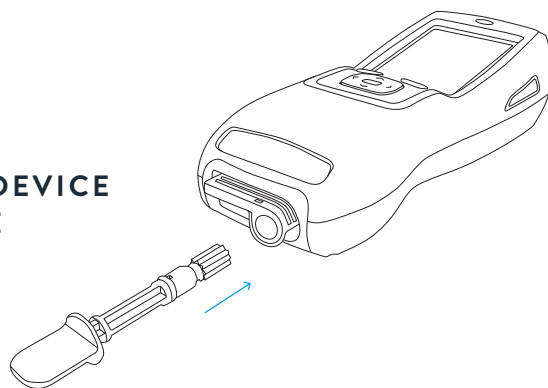

SYSTEM COMPONENTS



① TEST CARTRIDGE

- Quick response (QR) code contains lot and expiry information
- Test up to 6 drug classes in 1 convenient cartridge

② PRINTER

- Print test reports at point of test or from stored results

③ COLLECTION DEVICE

- Collect samples in 60 seconds or less
- The indicator will turn blue to indicate sufficient sample has been collected

④ HANDHELD ANALYSER

- Supports multiple languages
- LCD colour screen is readable day and night
- Intuitive and easy to operate

Actionable test results for 6 drug classes in **5 minutes**

PLEASE CONTACT YOUR LOCAL
ABBOTT REPRESENTATIVE TODAY
TOXICOLOGY.ABBOTT

1. Based on data from a non-drug using population.

© 2023 Abbott. All rights reserved. All trademarks referenced are trademarks of either the Abbott group of companies or their respective owners. Any photos displayed are for illustrative purposes only. Any person depicted in such photos is a model. COL-20376 v1 06/23

